

## SAFE "OFF - ON" FORCES

### 1. B. Procedure - continued

#### 4. Set the push - pull gauge to zero. (Fig. 4 - 12)

- Hold push - pull gauge at the same plane that you intend to use it.
- Put the red selector in the center position.
- Turn the dial on the face of the gauge until the zero lines up with the internal pointer.
- Push the red selector toward the end of the gauge you intend to push with. (Fig. 4 - 13) The gauge is now ready to use.

Figure 4 - 12

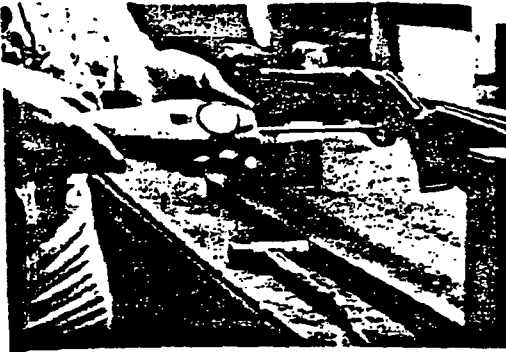


Figure 4 - 13

